



Quick and Easy **STEP-BY-STEP** INSTRUCTIONS

SELF-LIGATING PIVOT SETUP


STEP 1:

Using our exclusive SL tool, pull the door to the open position.



STEP 2:

Place the SL pivot onto the archwire and the push door closed with index finger.

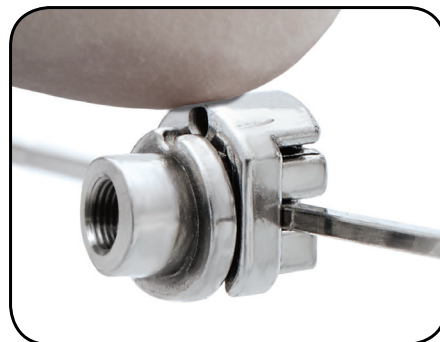

STEP 3:

Place CS4® spring onto pivot and secure in place with hybrid screw.



CLASS II



CLASS III

****The SL Pivot should be used on wire sizes from .016 x .025 up to .021 x .025**

IMPORTANT NOTE:
Active screw must be used with SL pivot.