


# Innovative technology of mixing materials produces a **glass like appearance...**

Introducing the new **Krystal™ SL Bracket System**! The **Krystal™ SL** is manufactured using a combination of copolymers and ceramics which create a translucent **glass-like material** for superior cosmetics and smooth finish for comfort. With torque in the base for excellent control, the clear clip provides enhanced aesthetics. Mechanical locking base with grooves allows for adhesive to be distributed easily along the bracket base. Bracket base is formulated to minimize breakage during debonding. This new innovative bracket has a **super smooth surface** which will not stain, break or flex during treatment. Easy to open and close for treatment, dependable, clear and comfortable, making the Krystal SL the perfect choice for both practices and patients!

## **Krystal™ SL** Self-Ligating System

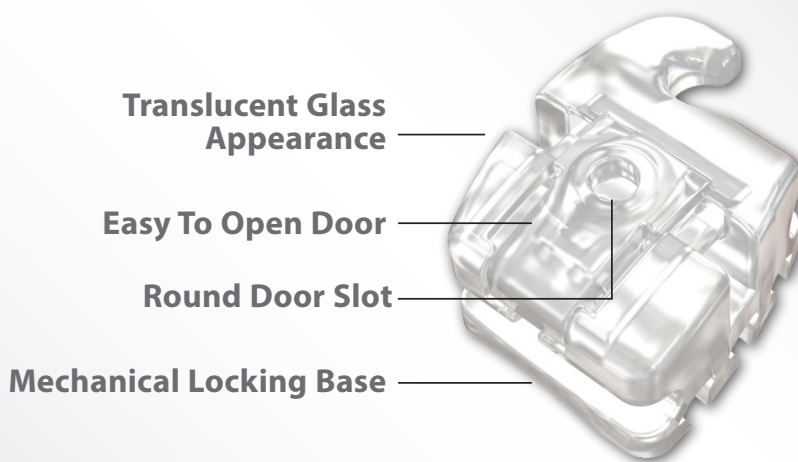

**Easy to use!** Place a tweezer or explorer onto the bracket, making sure the tool fits exactly in the door slot, pull the tool smoothly from the gingival to the occlusal and the door will open. Close the door using tweezers, tool or your fingers. Door is complete closed when you hear a "click".

 **DynaFlex®**

web: [www.dynaflex.com](http://www.dynaflex.com) ph: **800-489-4020**

082814 © 2014 DynaFlex®, St. Louis, MO 63074. Printed in U.S.A. All rights reserved. It is a violation of copyright law to reproduce all or part of this material, including photography, without the permission of DynaFlex®.