


# Care Stream Export Instructions



1. Open **CS OrthoTrac Software** and locate the patient.
2. Right click the **"Interoral 3D Models"** option and select **"Export to STL"** from the left-hand panel.
3. Select your export location, i.e. desktop, and **pair the STL files** with the filled out RX.

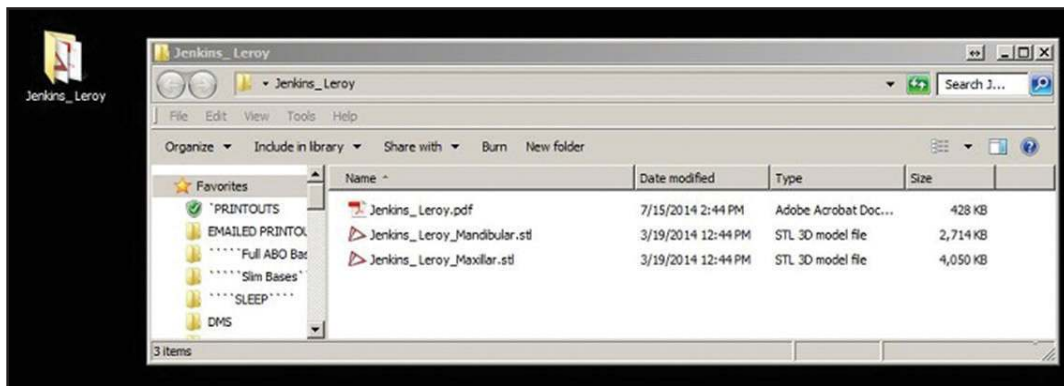

4. Pair the Rx together with the STL files in a folder with the **patients name** (last name, first name).

5. Double click the shortcut to the **DynaFlex® Egnyte Portal** or go to [DynaFlex.egnyte.com](http://DynaFlex.egnyte.com) to upload your case.



For instructions on how to submit your completed case,  
view the [Submit Your Case PDF](http://www.dynaflex.com/3d-ortho/) online at [www.dynaflex.com/3d-ortho/](http://www.dynaflex.com/3d-ortho/)