



Quick and Easy

# STEP-BY-STEP INSTRUCTIONS


## TRADITIONAL PIVOT SETUP

### CLASS II

### CLASS III


#### STEP 1:

Place 2 pivots on the upper archwire, position pivots distal to the upper canines, insert and ligate archwire

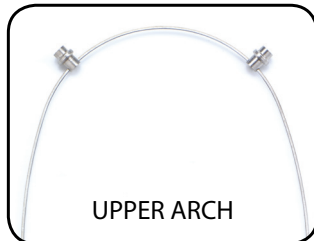

#### STEP 1:

Place 2 pivots on the lower archwire, position pivots distal to the lower canines, insert and ligate archwire

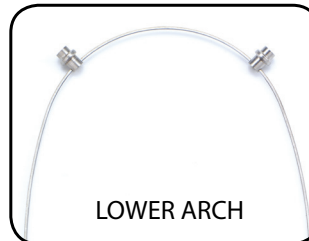

#### STEP 2:

Outside the mouth, assemble a spring, screw and pivot combination. Place assembled components onto the *lower* archwire. Insert and ligate archwire positioning the pivots mesial to the *lower* first molars



#### STEP 2:

Outside the mouth, assemble a spring, screw and pivot combination. Place assembled components onto the *upper* archwire. Insert and ligate archwire positioning the pivots mesial to the *upper* first molars



#### STEP 3:

Using a hemostat, stretch spring eyelet onto the upper pivot and secure in place with a screw



#### STEP 3:

Using a hemostat, stretch spring eyelet onto the lower pivot and secure in place with a screw

