


1. ARCH FORM

Check the arch form for accuracy (See below).

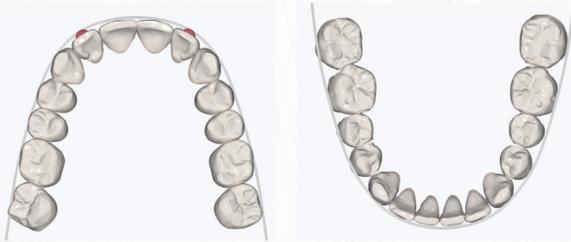


2. OCCLUSAL PLANE

Check that the occlusal plane has been leveled (See below).



3. MARGINAL RIDGES

We start with a view from the terminal molars looking towards the canines. We inspect that the marginal ridges line up from the molars, bi-cuspids and cuspids on both sides of the arch (See below).

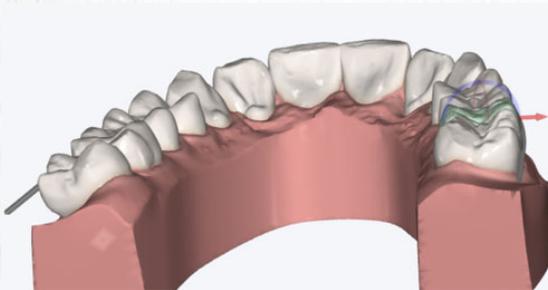


4. FOR ALIGNER REVIEW IPR RECOMMENDATIONS AND ATTACHMENTS

Review that the IPR recommended has eliminated all collisions. (See below).



Review attachment allocation for rotation, extrusion and retention (See below).

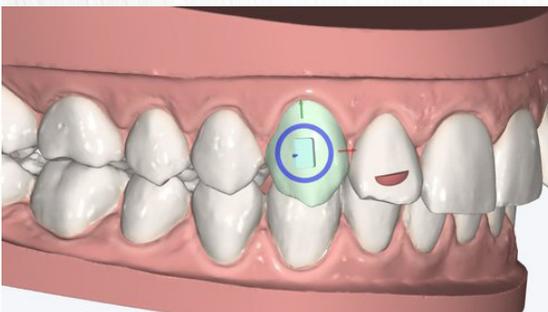

5. APPROVING THE SETUP

Once you are satisfied with the setup, Click **APPROVE** button to open approval dialog window (See below).

Approve



INSTRUCTIONS: QC CRITERIA / APPROVE SETUP

Approval dialog window will automatically appear with your User Name. To complete the process, enter your Password and click **Sign** (See below).

A screenshot of a 'Sign Order' dialog window. It features a title bar with a close button (X). Below the title bar, there are two input fields: 'User Name' containing 'staceyg@dynaflex.com' and 'Password' which is empty. At the bottom, there are two buttons: 'Sign' (highlighted in blue) and 'Cancel'.

6. REQUESTING SETUP REVISIONS

To request a SmileShare™ technician to perform a revision to the setup, Click on **REVISION** button (See below).



Enter revision notes in the revision comments box. (See below). Next, click **SEND COMMENTS** button to open revision request sign dialog window.

A screenshot of a 'Revisions' dialog window. It has a title bar with a close button (X). The main area contains a large text input field labeled 'Revision Comments'. Below this field is a blue button labeled 'SEND COMMENTS'. At the bottom right, there is a label 'Revision Notes'.

Revision dialog window will automatically appear with your User Name. To complete the revision request, enter your Password and click **Sign**. (See below).

A screenshot of a 'Sign Order' dialog window, identical to the one shown in the first image. It contains the 'User Name' field with 'staceyg@dynaflex.com', an empty 'Password' field, and 'Sign' and 'Cancel' buttons.